
**Brugsvejledning/Directions for use /
Gebrauchsanweisung / Bruksanvisning**
Kombi tilt / Combi tilt / Kombi-Stuhl mit Sitzkantelung/Komplett Mobil

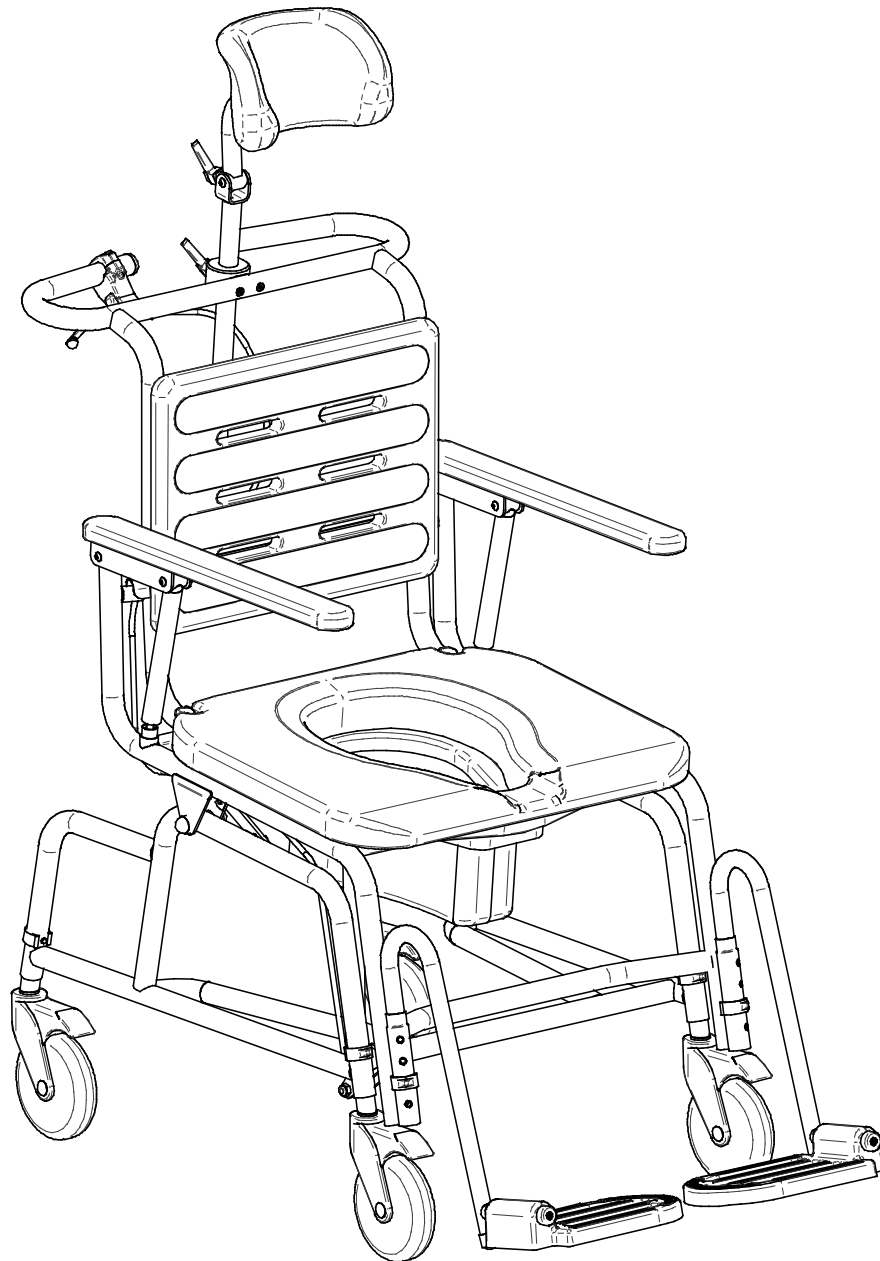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DIRECTIONS FOR USE COMBI TILT CHAIR

Please read the instructions before you start using this product

These instructions apply to the Combi Tilt Chair with variable tilting backrest function of 5 deg. forwards to 25 deg. backwards. The complete Combi Tilt Chair is made from rustproof materials.

PURPOSE

The Combi Tilt Chair is a very versatile chair. It can be used as a commode, a shower chair, and a transport chair for indoor transportation. When using the chair for patient transfer it is advisable to adjust the chair to the correct height and to fold the armrests up. Because of the chair's tilting function the Combi Tilt Chair has especially advantages when bathing the patient, when maintaining personal toilet hygiene and when patient transfer is performed by means of a hoist. In addition, patients who are not able to sit securely in an upright position can be transported more securely in a slightly backwards tilted position. The seat, backrest, and armrests are made of black polyurethane, a padded material with good anti-skid properties.

ASSEMBLY

Backrest

Remove the snaplocks from the frame and mount the backrest into the holes in the frame. Replace the snaplocks making sure that they are correctly inserted into the tubing.

Bucket frame / holder. The bed pan / bucket frame can be used for key hole shape bucket and round shape bucket. The two end pieces (B) rest on the rear of the seat frame. For key hole shape bucket, insert the two front rods (A) into the inner holes (C). For all other bucket shapes use the outer holes (D). If no bucket or bed pan is being used, then the frame can be detached.

Footrests

Footrests: start by moving the snaplock clip to the second last hole in the footrest tubing in the frame. Then slot the footrests into the tubing.

Castors: make sure the red dot is facing outwards & the castors are all at the same height.

INDIVIDUAL ADJUSTMENT

Height adjustment: By means of the snaplocks on the legs the Combi Chair is height adjustable. Remove the snaplocks one by one and select one of the height adjustment positions. Make sure that the four legs have the same height and that all of the snaplocks are inserted correctly into the tubing again.

Headrest The headrest is height, angle and laterally adjustable by loosening the hand levers. Tighten the levers securely afterwards.

Tilting function:

Before using the tilting function remember to lock all four castors. The chair is tilted by means of a gas piston activated via a hand lever on the pushbar.

Rear operation: The helper must use the pushbar to control the backward and forward tilting movement.

Side operation: When tilting the chair forwards, the helper places one hand on the pushbar and one hand on the seat surface.

Please note that Combi Tilt requires a counterweight in the chair in order to tilt the chair forward easily, ie. it requires someone to be seated in the chair. Attempting to tilt the chair to a forward position without a counterweight in the chair will require greater physical exertion from the caregiver.

The chair can be tilted 5 deg. forward and 25 deg. backwards.

MAINTENANCE

To be wiped clean after use. Clean the Combi Chair with domestic cleaning materials without abrasive or corrosive properties. Solvents and chlorine should not be applied.

Seat and backrest can be detached for cleaning purposes. The polyurethane material can tolerate temperatures of up to 70°C. Make sure to reattach all removed parts correctly. The bucket holder is easily cleaned when removed.

WARRANTY

2 Years against frame breakage / 7 years against rust damage

CAUTION

- Max. load: 150 kg. / 330 lbs
- When the Combi Tilt Chair is not used for transportation purposes or in the tilting function, all four castors must be locked.
- The backrest must only be removed if the chair is being transported without the patient. The chair must not be tilted if the backrest is removed.
- The chair must only be tilted by the caregiver and not by the user.
- Armrests: Caution must be exercised when folding up the armrests as there is a risk of ones' fingers getting caught between the upright frame tubing of the backrest and the upright telescope tubing of the armrest.
- Check the condition of the snaplocks frequently. Regularly check that the snaplocks are inserted correctly into the tubing.
- If the castors are removed for any reason or if the chair is height adjusted it is important that the castors are correctly mounted again and the snaplocks properly inserted. Make sure the silver plates are facing outwards / the front of the wheel.
- When removing the bucket frame, follow the instructions as stated in " Assembly"
- Regularly check that the cable to the gas piston has the correct tension."

Technical Specification:

Height:	950-1180 mm
Length excl. footrests:	870 mm
Længde incl. footrests:	1090 mm
Seat height:	525-620 mm
Width:	535 mm
Width between armrests:	430 mm
Length of armrests:	480 mm
Width of armrests:	50 mm
Backrest:	410 x 310 mm
Seat width:	510 mm
Seat depth:	435 mm
Seat hole width:	210 mm
Seat hole depth:	310 mm
Castors:	125 mm, incl. brakes
Max. weight load:	150 kg

Materials:

Frame:	Stainless steel
Surface coating frame:	Powder coating
Seat, Backrest, Armrests:	Polyurethane
Footplates:	Polypropylene
Snaplock Clips:	Brass/Polypropylene

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